Accessori | Accessories

Beaker in pirex (conf. 2 pz.)
Pyrex beaker (Pack 2 pcs.)

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EN

The ultrasonic tub is designed to **wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing**. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

Heating

The device proceeds to the heating phase of the detergent. It stops automatically when the preset temperature is reached. Always consider that during the ultrasonic cleaning the temperature of the detergent tends to rise independently.

Degassing (DEGAS)

The device starts activating the ultrasonic phases intermittently. This process reduces the gas dissolved in the liquid, thus improving the efficiency of the subsequent washing.

Ultrasonic cleaning

Washing phase through ultrasonic cavitation.

SWEEP

Ultrasounds are modulated in frequency, thus reducing the negative effects of standing waves, increasing the distribution of ultrasonic energy within the washing liquid. The cleaning is more efficient (**improved cavitation**).

Discharge

The tub is emptied by opening the appropriate valve.

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EN

Tecno-Gaz S.p.A. has been able to create **a new and revolutionary device that can automatically carry out all the major preparation phases for the sterilization cycle, such as disinfection, cleansing, rinsing, drying, without human intervention, without any professional risk and with a reduction in the overall time taken.** A device that is unique in the world and indispensable for every facility that carries out sterilization.

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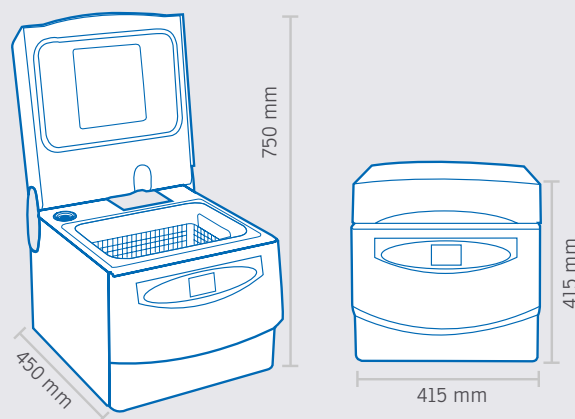
The aspects that Multisteril Fast can help you manage correctly

Assistants' expenditure of time

Assistants spend a lot of time for the preparation phase to tools sterilization, because of multiple and complex manual procedures. **Multisteril Fast does not require human presence.**

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Risk factor

The manipulation of tools and potential infectious materials is a very important factor and it must be kept under strict control, both for operators' protection and the responsibility of the owner of the studio. **Multisteril Fast cancels the risk.**

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Management protocol

There is the sterilization if the procedure of preparation of materials is done properly, if not, the whole process is completely nullified. **Multisteril Fast performs correctly all the cycles.**

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Tested disinfection

Tested issued by University of Dresden attesting cleaning **capacity of Multisteril Fast in combination with Multisteril CD cleaner.**

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Spaces

Space is another important problem, in fact, sterilization rooms often have small size and rationalization of space assumes a considerable value. **Multisteril Fast can do all the process filling 40 cm.**

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Overall costs

Cost is an important component which must be brought under control. **With Multisteril Fast every cycle costs about 2 Euros.**

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Liquid
concentrate for
the non
protein-fixing
cleaning and
disinfection

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Liquid concentrate for the non
protein-fixing cleaning and disinfection

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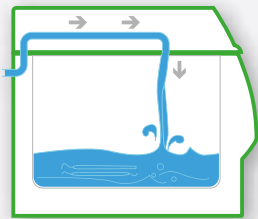
Support for 2 surgical trays
(article code refers
to support only)

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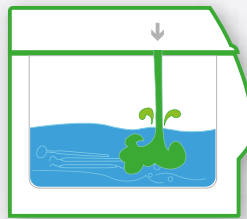
Loading the water

The water enters the tank thanks to the **direct connection to the mains water supply.**

Stand-by

To improve cleansing, an initial standby phase has been created, which allows the tub to be filled with water and liquid detergent/disinfectant, so that the instruments sit better in the solution, **ensuring perfect cleansing, even when using bi-component cements.**

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
Disinfectant/detergent entry and liquid activation

A special peristaltic pump present inside the device automatically **introduces the percentage of disinfectant/detergent previously outlined in the parameters of the cycle.**

Fasi del processo | Phases of the process

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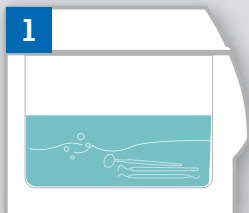Heating

The device moves on to heating and to a **first static decontamination of the instruments.**

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Degasification

This process **eliminates the gases dissolved in the water** avoiding interference with the ultrasounds and improving efficiency.

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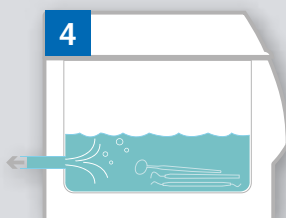
Ultrasonic wash

At the end of the degasification phase the ultrasonic cleansing begins, setting off the cavitation, which allows for the removal of chemical and organic residue present on the materials immersed in the tank, **destroying the bacterial load by more than 95%. The length of this procedure can be programmed by the operator for between 5 and 30 minutes.**

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
Waste with pump

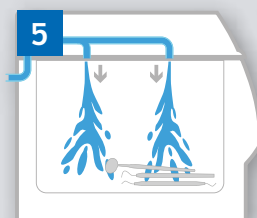At the end of the cleansing phase, **MultiSteril automatically empties all the liquids present in the tank.** Once the tank is completely empty the following rinse phase begins.

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
Rinse

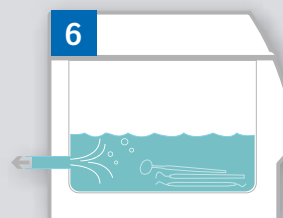To remove all chemical residue from the instruments, **the rinse phase begins, taking place through the pressurised intake of water through the special nozzles in the lid.** The water taken in is then automatically removed through the mesh with a special pump that permit to make the process faster. At the end the drying phase is launched.

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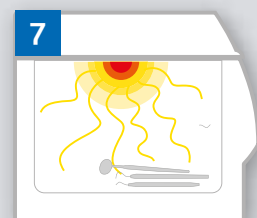
Drying

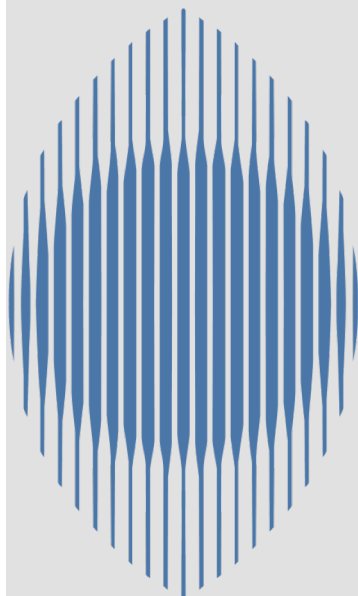
To guarantee a proper drying of the materials **the machine generates a flow of hot air, produced by a fan and a resistor located on the lid of the device,** which lightly touches the objects and takes away the dampness in the form of gas.

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SIR Oftalmica Srl